

# OSOM<sup>®</sup>



## Strep A Test

*Clear, accurate results in just minutes can help everyone feel better before the office visit is over.*



✓ **VERY ACCURATE**

96% sensitivity; 98% specificity.

✓ **FAST**

Results in 5 minutes.

✓ **EASY-TO-READ**

Two-color results: blue test line, red control line.


✓ **CONVENIENT**

Compact packaging. Everything you need to run 50 tests. Kit Includes positive and negative controls. Room temperature storage.




**TEST PROCEDURE / RESULTS**

Add 3 drops Reagent 1 and 3 drops Reagent 2 to the test tube.

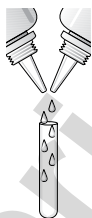

Insert swab and rotate 10 times vigorously.

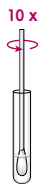


10 x

Let stand for 1 minute.



1 MIN

Express liquid from the swab by squeezing the sides of the tube as the swab is withdrawn.

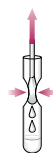

Place the test stick into the extracted sample.



5 MIN Read results at 5 minutes. Positive results may be read as soon as red control line appears.



**POSITIVE**



**NEGATIVE**