

## How do I install a Pre-Calibrated Alcohol Sensor Module?

For the **AlcoMate Prestige**, please take the following steps to install a new sensor module (Click on the images for larger versions):

		
Prepare your AlcoMate Prestige.	Use a flat, sharp tool to carefully open the top cover.	Lift the breath tube to access the sensor module.
		
Remove the breath tube. It should remove easily.	Carefully raise the sensor module from the body. Do NOT detach the cable from the body.	Remove the cable from the slot underneath the sensor module.
		
Prepare a new pre-calibrated sensor module.	Insert the cable into the new sensor. Make sure the blue strip is toward the sensor, as shown.	Replace the sensor module into its original position.
		<p><b>Ready for Testing!</b></p>
Re-attach the breath tube directly on to the sensor. Make sure the front opening is aligned.	Re-attach the cover on to the top of the body. It should snap in fairly easily.	

