



Wouldn't you want to know?

FACT: 50-60 million people are infected with genital herpes (HSV-2) today in the United States.

FACT: 90% of them don't know they have HSV-2.

Who's at Risk?

- Any patient at risk for sexually transmitted diseases (STD) including HIV.
- People who have a history of symptoms suggestive of genital herpes.

About Genital Herpes

Genital herpes is a sexually transmitted disease caused by the herpes simplex viruses type 1 (HSV-1) and type 2 (HSV-2).

Over 80% of recurring genital herpes is due to HSV-2. Type-specific serology has been shown to be the best way to detect persons with subclinical or recurring genital herpes.

The CDC recommends that at risk patients be offered type-specific serology tests to accurately distinguish between HSV-1 and HSV-2 infections*.

Once HSV-2 is diagnosed by testing, therapy can suppress HSV-2, reduce painful outbreaks, and decrease the risk of transmitting HSV-2 to a partner. Testing and treatment may also reduce the risk of acquiring HIV or experiencing HIV progression.

biokit[®]HSV-2^{**}

Rapid Test

**A type-specific gG-2 immunoassay kit,
a leading aid in the diagnosis of genital herpes.
Only glycoprotein G (gG-2) based assays have been
shown to be accurate in detecting antibodies to HSV-2^{***}**

biokit[®]HSV-2 Rapid Test Kit Features

- Type-specific serology test recommended by experts
- Finger-stick blood or serum specimens
- 7-10 minute test time
- Easy to use

Two-reagent system

Built-in controls

- Sensitivity – 96% (compared to viral culture and Western blot)^{****}
- Specificity – 98% (compared to viral culture and Western blot)^{****}
- Seroconversion in as little as two weeks
- Designed for use in Physician's Offices, Laboratories, Family Practice and STD clinics

Product Information

- Packaging – 20 tests per kit
- Product Number – 3000-28001
- CPT Code – 86696

* "CDC Sexually Transmitted Diseases Treatment Guidelines; May 10, 2002

** Formerly POCKit HSV-2

*** *Morrow, Rhoda A, Friedrich David. Inaccuracy of Certain Commercial Enzyme Immunoassays in Diagnosing Genital Infections with Herpes Simplex Virus Type 1 or 2. Amer J Clin Pathol 2003;120:839-844.*


**** "Premarket Evaluation of the POCKit HSV-2 (currently **biokit**HSV-2) Type-Specific Serologic Test in Culture-Documented Cases of Genital Herpes Simplex Virus Type"; *Rhoda Ashley, PhD, Anna Wald, MD and Marie Eagleton, PhD; Sexually Transmitted Diseases; May, 2000, pages 266-269.*



Test Procedure




1 Label the device with the patient's name or identification number and peel back the protective label, revealing the round TEST WINDOW.

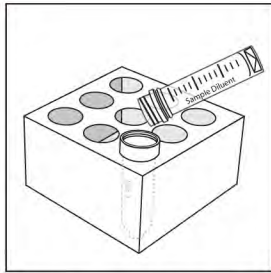


2 Place the empty plastic Vial 1 into a rack. Pour contents of the Sample Diluent vial into Vial 1 and discard the empty Sample Diluent vial.




3 Take the patient's blood sample, using the capillary pipet provided. When using serum (50µl) dispense directly into vial.

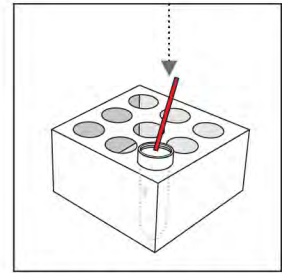


4 Place the blood-filled capillary tube into Vial 1 which contains the sample diluent buffer.

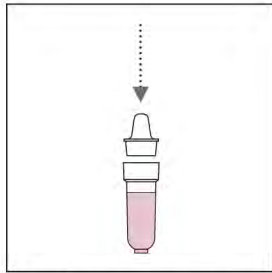


5 Fit the pre-filter tip onto Vial 1, ensuring that the tip clicks securely into the vial. Gently mix the blood sample and diluent by inverting Vial 1 several times.

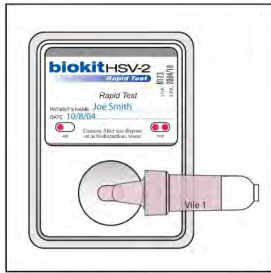

6 Keeping the pre-filter tip on the vial, squeeze all the sample/diluent mixture onto the TEST WINDOW, by squeezing the middle of the vial.

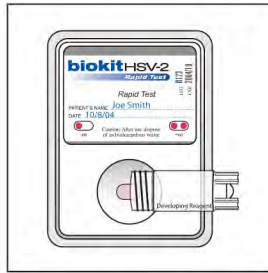

7 Allow the sample diluent mixture to be absorbed into the test device. Mix the developing reagent by inverting several times. Remove lid, dispense into the TEST WINDOW.

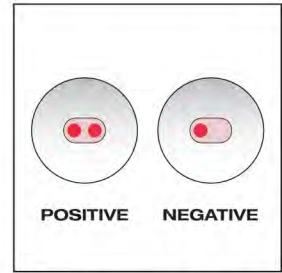

8 Read and record the results of the test.

Additional biokit Products

Monogen 3000-21001	50 Tests	Latex Test for Mononucleosis
Monogen 3000-21002	150 Tests	
RPR 3006-25511	100 Tests	Charcoal Test for RPR (Syphilis)
RPR 3006-25550	500 Tests	
Rubagen 3000-24001	100 Tests	Latex Test for Rubella Immunity
Rheumajet ASO 3000-27007	50 Tests	Latex Test for Anti-Streptolysin O
Rheumajet CRP 3000-27049	50 Tests	Latex Test for C-reactive Protein
Rheumajet RF 3000-27034	50 Tests	Latex Test for Rheumatoid Factor
Rheumajet RF 3000-27033	100 Tests	Latex Test for Rheumatoid Factor

To Order



6658 Youree Dr Ste 180 PMB 404
Shreveport, La 71105

www.americanscreeningcorp.com
Sales@americanscreeningcorp.com

Toll Free: 1-866-526-2873

Ph: 318-798-3306 Fax: 318-798-3386