

Multi-Panel Saliva Screening Kit



www.americanscreeningcorp.com

Oral Fluid Screening Kit

Drug testing made simple

- Observed collection, difficult to fake
- Simple 2 piece kit
- 18-24 month shelf life
- Easy to use

6658 Youree Dr Ste 180 PMB 404 Shreveport, La 71105

Toll Free: 1-866-526-2873 Ph: 318-798-3306 Fax: 318-798-3386 www.americanscreeningcorp.com

Multi Panel Saliva Screening Kit

This is a rapid screening test for the simultaneous detection of drugs in human saliva .


Benefits

- Simple 2 piece package
- Observed collection
- 18-24 month shelf life
- Lowest industry price

Two options available in stock:
 THC/COC/OPI/AMP/MAMP/PCP
 THC/COC/OPI/MAMP/BZO/OXY




COLLECTION PROCEDURES

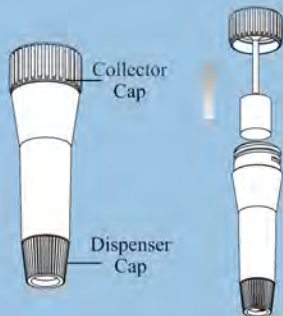

1. Bring the pouch to room temperature, remove test and use as soon as possible. Unscrew the collector cap and remove the sponge.




2. Insert the sponge in the mouth and swap for 3 minutes. The sponge will be free of hard spots when enough oral fluid is collected.

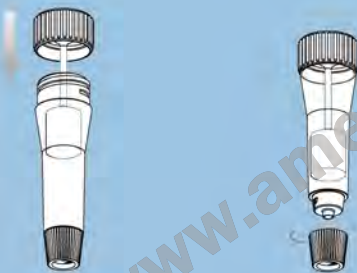

3. Place the testing device on a level surface and insert the sponge back into the collection tube.

4. Unscrew the dispenser cap.

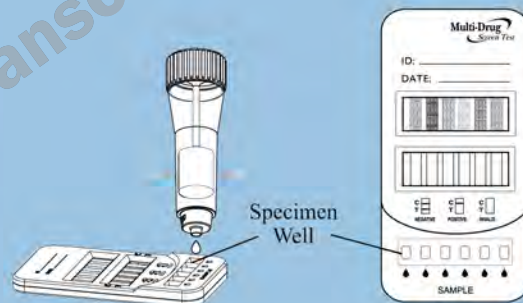

5. Squeeze the tube toward the specimen test well. Place 1 drop of fluid in each well, then read results in 5 minutes.



NEGATIVE



POSITIVE



INVALID

TEST IS DESIGNED AS SINGLE USE ONLY