




Discover Key Cup Procedure Card

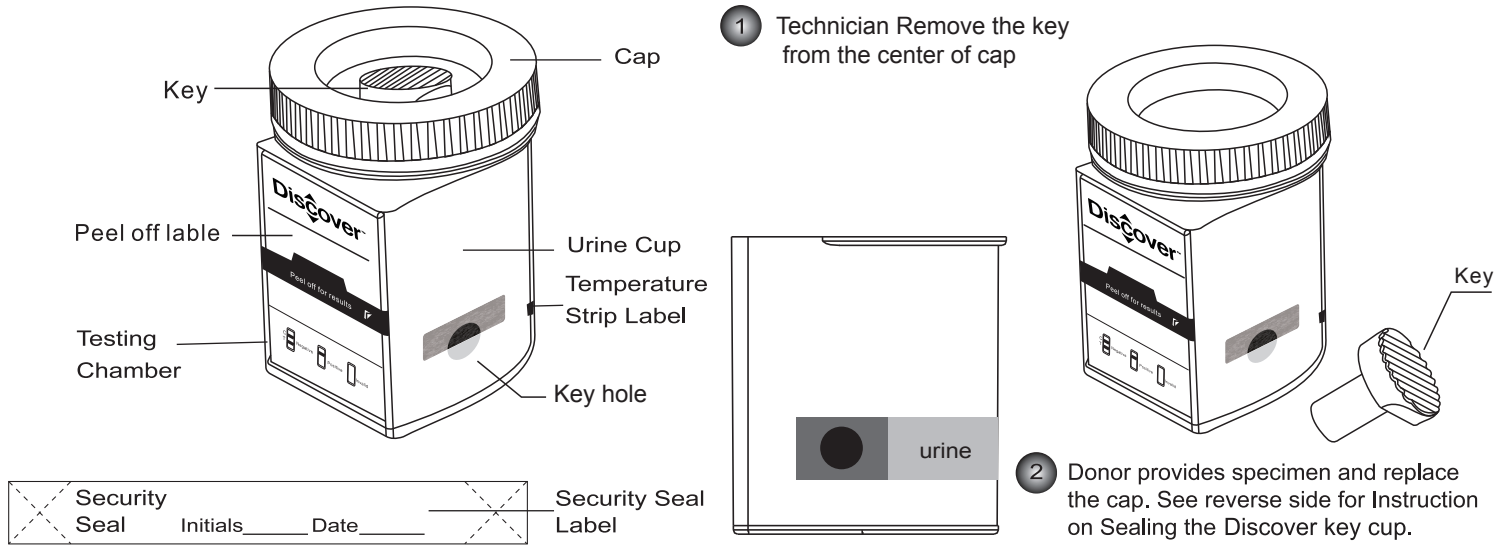

- 3** Read urine temperature between 2-4 minutes after voiding. Normal urine temperature should fall the ranges from 90°F to 100°F.
- 4** Place the cup on a flat surface, insert the key into the key hole of the cup, then push the key to a fully closed position to initiate the test and then start the timer.
- 5** Technician peels off the label to view the results.

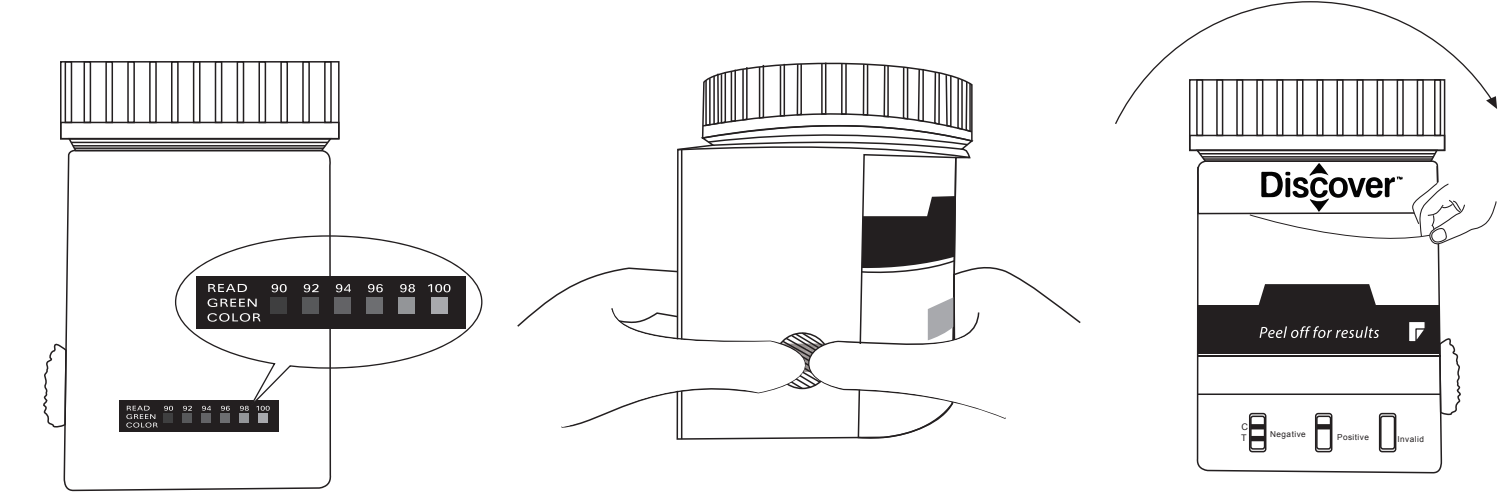


- 6** If adulteration test is included on the test cup, read the adulteration test results between 2-5 minutes.
- 7** Read the drug test results at 5 minutes.
- 8** A photocopy of the results can be made by placing the entire cup with the results face down on the copier.

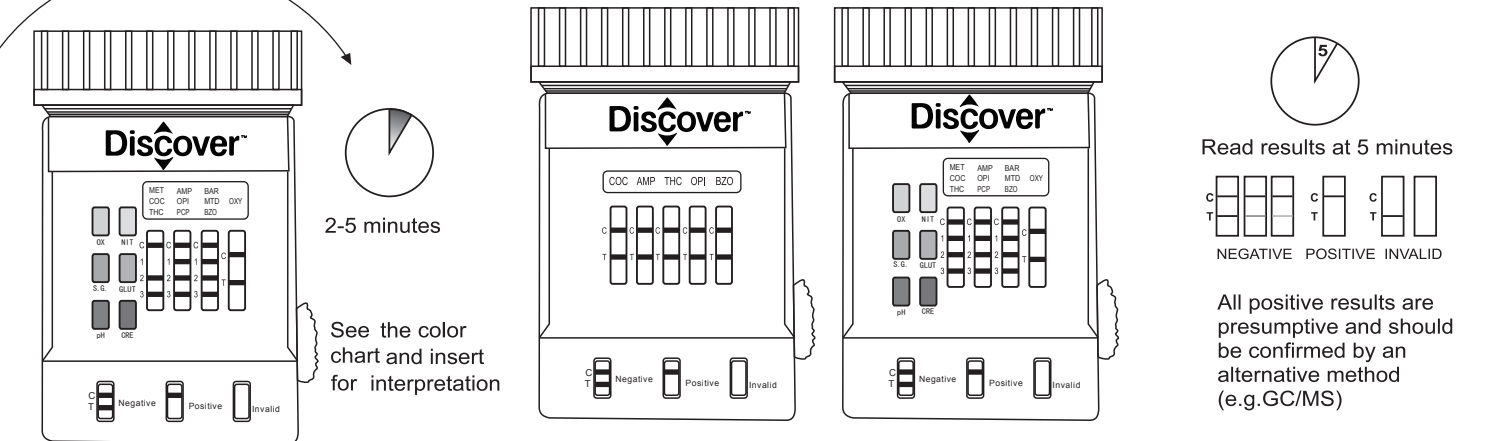

Example: Test shows positive for MET on a 10-panel Cup.
 (See reverse side for Instruction On Sealing Discover key cup With The Cap)

Instruction On Sealing Discover Key Cup With The Cap


- 1 Pour the urine sample into the cup

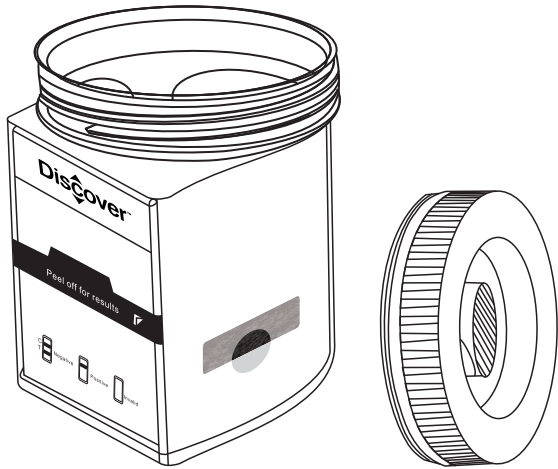


- 2 Place the cap flat on top of the cup




- 3 Twist the cap clockwise until completely tightened

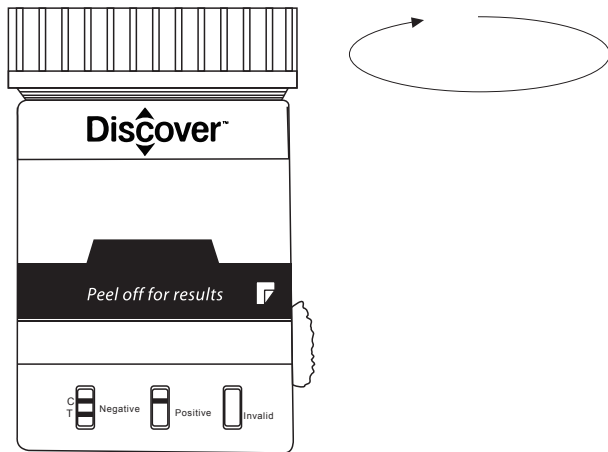

- 4 Check to make sure the cap aligns with the top of the cup and is completely sealed shut

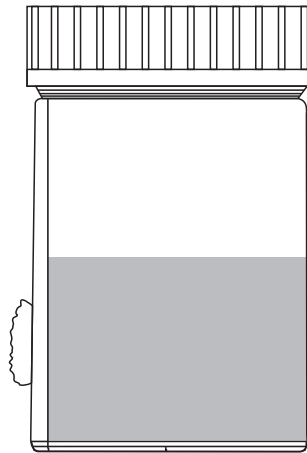

- 5 If the test result is positive, secure the cap with the Security Seal, and send the cup to laboratory for confirmation testing

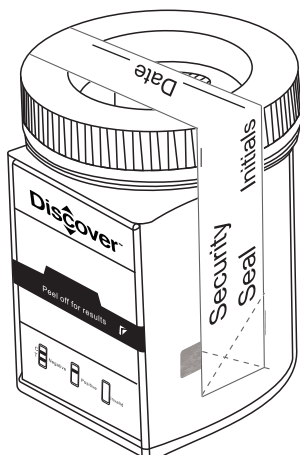

Manufactured for:
American Screening LLC
9742 St. Vincent Ave, Ste. 100,
Shreveport, LA 71106, United States

sales@americanscreeningcorp.com
www.americanscreeningcorp.com
Customer Service Phone: 1-318-798-3306
Service date/hours: Monday through Friday 8:00 AM to 5:00 PM CST

