


SwabScan Oral Fluid Drug Tests Procedure Card

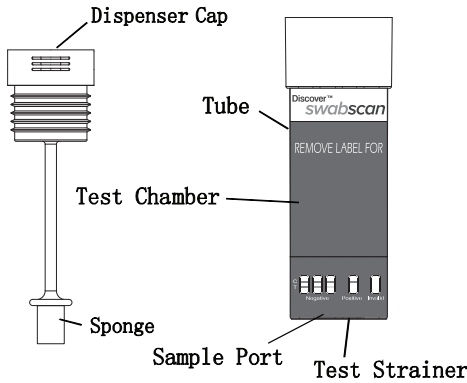

1. Open the sealed pouch, then remove the Saliva collector from the sealed plastic bag.


Allow the test tube, specimen, and/or control to reach temperature (15-30°C) prior to testing.

2. Donor insert the sponge into the mouth, actively swab the inside of the mouth and tongue to collect oral fluid for approximately 3 to 5 minutes until the sponge becomes soft and fully saturated. The sponge will be free from hard spots when fully saturated.




Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.


3. Remove the collector from the mouth, put the saturated oral fluid collector into the strainer of the test tube (on the central of the bottom).

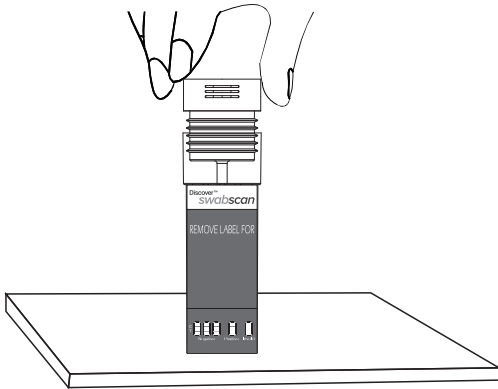


4. Keep the test tube upright and push the cap into the tube vertically. Place the test tube on a flat surface and start the timer.

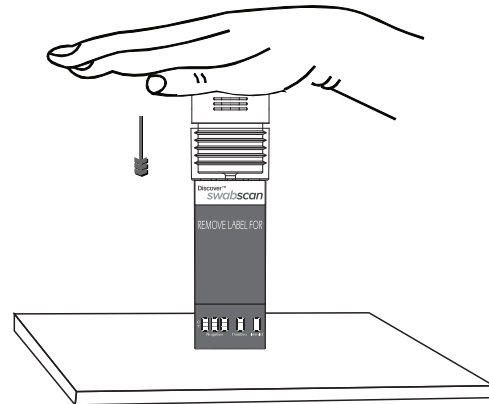

5. Technician peels off the label to view the results.

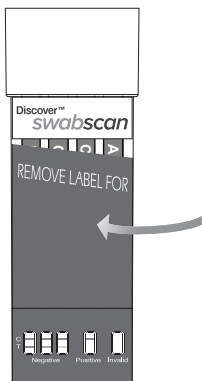

6. Wait for the colored line(s) appear. Read the results of drug tests at 5 to 10 minutes, read the alcohol test result at 2 to 5 minutes. Do not read the results of drug tests after 1 hour, do not read the alcohol test result after 5 minutes.

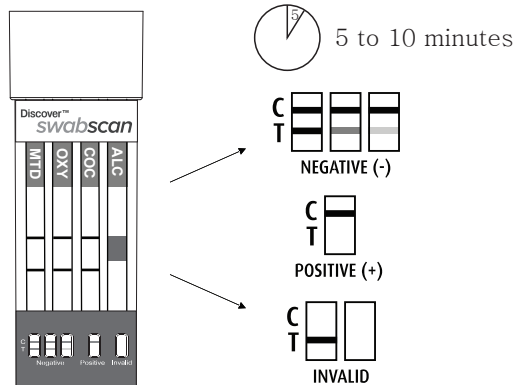

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)