

Discover Multi-Panel Drug Test Cup Procedure Card

1 Remove the cap.

Cap
Peel Off Label
Urine Cup
Temperature Strip Label

2 Donor provides specimen and replace the cap. See reverse side for Instruction on Sealing the Multi-Panel Drug Test Cup.

Security Seal Initials _____ Date _____ Security Seal Label

- 3** Read urine temperature between 2-4 minutes after voiding. Normal urine temperature should fall the ranges from 90°F to 100°F.
- 4** Technician secures the cap tightly while the cup is on a flat surface, and dates and initials the security seal and places the security seal over the cap.
- 5** Technician peels off the label to view the results.

- 6** If adulteration test is included on the test cup, read the adulteration test results between 2-5 minutes.
- 7** Read the drug test results at 5 minutes.

6 If adulteration test is included on the test cup, read the adulteration test results between 2-5 minutes.

7 Read the drug test results at 5 minutes.

8 Read results at 5 minutes

2-5 minutes
See the color chart and insert for interpretation

Drug Test Name(s)
C Line in "C" Region
T Line in "T" Region

NEGATIVE (-) POSITIVE (+) INVALID


All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)

Example: Test shows positive for THC.
(See reverse side for Instruction On Sealing Multi-Panel Drug Test Cup With The Cap)

Instruction On Sealing Multi-Panel Drug Test Cup With the Cap



1 Pour the urine sample into the cup

2 Place the cap flat on top of the cup

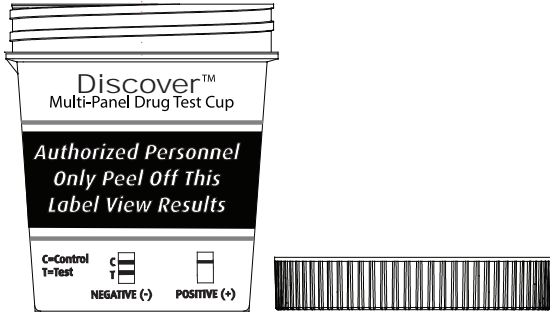

3 Twist the cap clockwise until completely tightened

4 Check to make sure the cap aligns with the top of the cup and is completely and tightly closed

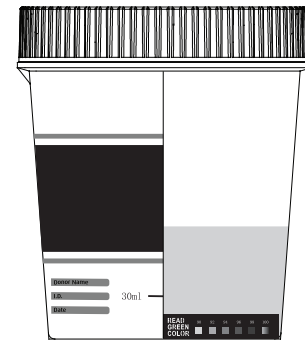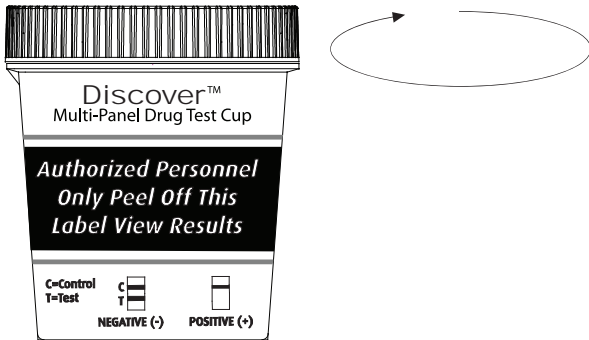

5 If the test result is positive, secure the cap with the Security Seal, and send the cup to laboratory for confirmation testing

