


# osom<sup>®</sup>

## Ultra Flu A&B Test


### NASOPHARYNGEAL WASH/ASPIRATE SAMPLE PROCEDURE (PURCHASE OF 1006 REQUIRED)

#### INTENDED USE


The OSOM Ultra Flu A&B Nasopharyngeal/Wash/Aspirate Kit is intended for use in testing with the OSOM Ultra Flu A&B Test. Refer back to package insert for full instructions and use.

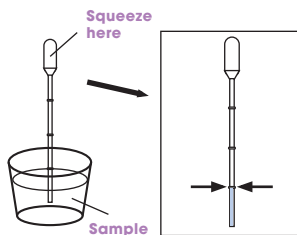


1. Draw nasal wash or nasopharyngeal aspirate sample to the first (lowest) mark of the graduated transfer pipette.

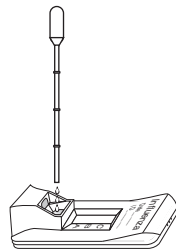


2. Dispense the entire sample in the transfer pipette into the Extraction Well of the test device.

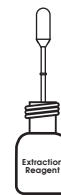


3. Remove the cap from the Extraction Reagent bottle.
4. Using a new transfer pipette, draw Extraction Reagent Solution to the first (lowest) mark.

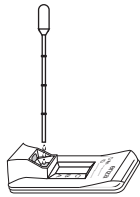


5. Dispense all of the solution in the transfer pipette into the Extraction Well of the test device.

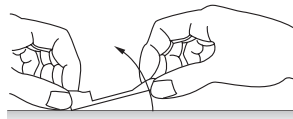

6. Incubate 1 minute. Re-cap the Extraction Reagent bottle.

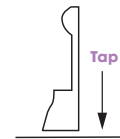

7. Raise the test device upright (see picture).
8. Let it stand for 1-2 seconds. Tap the device on a flat surface to ensure that the liquid flows into the hole.

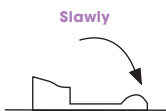

9. Immediately after tapping, slowly lay the device back down onto the flat surface.



10. Start timing. Read test results at 10-15 minutes. Confirm negative results at 15 minutes.

**AmericanScreening™**



American Screening LLC  
9742 St. Vincent AVE  
Suite 100  
Shreveport, LA 71106  
1-318-798-3306

**ASSISTANCE**  
For assistance, call ASC  
Technical Support at  
3187983306 (U.S. only)